# Melilax

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EU PATENT GRANTED

EP2919857B1

ACTIVE COMPLEX OF HONEYS AND POLYSACCHARIDES FROM ALOE VERA AND MALLOW

**Easy Use**  
ergonomic squeeze bottle  
+ easy to apply

**Double Action**  
**Promotes** bowel movement without irritation,  
**protecting** the rectal mucosa

gluten free

Suitable for coeliacs

## MEDICAL DEVICE Class IIb

**EN** Melilax is an innovative **micro-enema produced with honey**, which combines a balanced evacuative action with a protective and soothing action on the rectal mucosa to help relieve the discomfort, irritation and inflammation caused by constipation.

**The double action of Melilax** is achieved thanks to **PROMELAXIN**, an active complex of nectar and honeydew honeys carefully selected and processed based on their monosaccharide, polysaccharide and melanoidin content and enriched with the **polysaccharide fraction** of Aloe Vera and Mallow.

### INDICATIONS

**Melilax** is indicated in the treatment of constipation, even when accompanied by visceral hypersensitivity (e.g. irritable bowel), fissures or haemorrhoids.

The product is also suitable for use during pregnancy, breastfeeding and in infants.


### HOW IT WORKS

The non-irritating evacuative action of **Melilax** generates a balanced, physiological stimulus to produce defecation. Furthermore, thanks to its viscosity and mucus-like properties, **Melilax** also protects the rectal mucosa during defecation, imitating the lubricating action of the physiological mucus.


These particular properties combined with its antioxidant action enable the product to protect and soothe the mucosa, through an indirect anti-inflammatory action derived from the physical characteristics of the product.

### INSTRUCTIONS FOR USE

1) Open the micro-enema by removing the safety cover: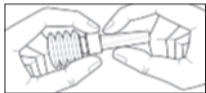

- Hold the ring above the squeeze bottle
- Using your other hand, bend the cannula cover until it breaks free from the ring

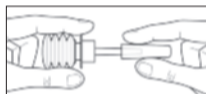- Remove the cannula cover

2) Apply several drops of the product in the perianal area and gently insert the cannula into the rectum.

3) Fully depress the micro-enema; after application, keep it depressed until the cannula is completely removed to avoid drawing the product back.

### WARNINGS

Do not use in case of individual hypersensitivity or allergy to one or more components. If constipation persists, see a doctor in order to exclude other medical conditions.

The micro-enema is disposable and should not be reused, even when half-doses are administered; for example, in infant use.

Store at room temperature, away from heat sources and protected from light. Keep out of reach of children. The expiry date refers to the intact product, stored correctly.

### COMPOSITION

#### Functional substances 73.2%

**Promelaxin**, active complex of honeys and Aloe Vera and Mallow polysaccharides

*Titrated in: monosaccharides ≥ 50%, polysaccharides (molecular weight > 20,000 daltons) ≥ 0.3%*

Also contains: hydroglycerin mixture, Lemon juice, essential oil of Lavender.

### DOSAGE

- **Adults and children over 12 years: 1 x 10 g micro-enema** as needed. In cases of obstinate constipation, apply 2 consecutive doses.
- **Children from 3-12 years: 1 x 5 g micro-enema** as needed. In cases of obstinate constipation, apply 2 consecutive doses.
- **Children from 1-3 years: 1 x 5 g micro-enema** as needed.
- **Infants from 0-1 years: half a 5 g micro-enema** as needed.

To facilitate micro-enema application, position the child on his left side with his knees tucked into the stomach.

### PACK FOR SALE

Melilax Adult: 6 x 10 g disposable micro-enemas with cannula cover for adults and adolescents

Melilax Pediatric: 6 x 5 g disposable micro-enemas with cannula cover for children and infants

### EXPIRY DATE

See the expiry date stated on the pack. Do not use beyond the expiry date indicated.

**Last revision date:** 09/2020

### FOR TODAY AND TOMORROW'S HEALTH

Aboca is a Benefit Corporation and certified B Corp. **With us, you contribute to a sustainable future.** Learn more about our commitment at [www.aboca.com](http://www.aboca.com)

The Melilax formulation is 100% biodegradable and contains no synthetic, semi-synthetic or genetically modified substances.

### PACKAGING MATERIALS

Carton box	PAP 21	>	Paperboard
Leaflet	PAP 22	>	Paper
Sachet	PP 5	>	Plastic
Squeeze bottle and Cannula	LDPE 4	>	Plastic
Cannula cover	LDPE 4	>	Plastic

For disposal, please check the regulations in your local area.

[www.melilax.com](http://www.melilax.com)

**Aboca S.p.A. Società Agricola** - Località Aboca, 20  
52037 Sansepolcro (AR) - Italia - [www.aboca.com](http://www.aboca.com)

Company operating under certifications ISO 9001 and ISO 13485 for quality management system and ISO 14001 for environmental management system.

Certified

Corporation

ALLENMEL-A