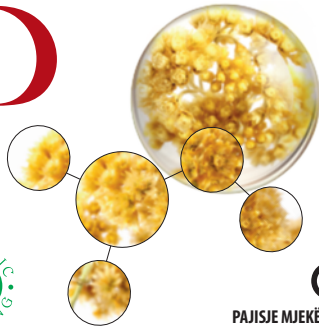
FitoroiD^{neo}

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
FitoroiD^{neo}

Haemorrhoidal discomfort
Soothes pain, burning and itching
while protecting the mucosa

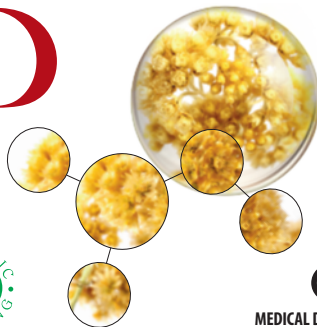With Helydol and Tea tree oil

CE 0373
MEDICAL DEVICE Class IIb

ENDORECTAL BIOOINTMENT

HOW IT WORKS: NeoFitoroid calms irritation, protects the mucosa and supports its normalisation through **3 mechanisms of action:**

- Mucoadhesive:** adheres to the mucosa, forming a barrier-like film that protects it from contact with irritants;
- Lubricant:** reduces the friction caused by stool passage to protect against irritation and perianal congestion;
- Antioxidant:** protects the mucosa against free radicals, thereby indirectly alleviating inflammation.

In addition, NeoFitoroid creates an environment that is **not conducive to the growth of microorganisms**, thus providing an additional action that protects and reduces itching. Its **refreshing properties** offer rapid relief to discomfort.

NeoFitoroid does not contain corticosteroids or anaesthetics, and its mechanism of action ensures that it does not irritate or dry out the mucosa.

INDICATIONS: indicated for the treatment of **haemorrhoids** (both internal and external) **and their symptoms**, such as **pain, burning and anal and perianal itching**. It can be used **even in the case of anal fissures**.

It can also be used **in prevention and in all situations** that can cause **irritation and/or congestion** of the anal and perianal region (constipation, diarrhoea, unhealthy eating or lifestyle).

Also suitable during pregnancy and breastfeeding.

INSTRUCTIONS FOR USE: for external application, gently apply and massage onto the anal and perianal area.

For internal application, use the endorectal cannula provided, which can be screwed onto the tube. Apply enough ointment to cover the affected area.

Use the product after bowel movements, following careful and gentle washing of the anal and perianal area. As a preventive measure against irritation caused by friction, the treatment can be carried out before bowel movements, remembering to reapply the product afterwards. After use, unscrew the cannula, clean it thoroughly with soap and warm water, and dose the ointment tube. It is possible to repeat the application several times a day, even at close intervals and for prolonged periods.

COMPOSITION - Functional substances: dried extracts of the lipophilic fractions of Helichrysum flowering tops (Helydol)* and Butcher's Broom root*; Gel of dehydrated Aloe Vera leaf*; Aqueous solution of Helichrysum flowering tops*; Oily extract of Hypericum flowering tops*; Jojoba oil*; Shea butter*; Tea tree, Cypress and Peppermint essential oils. *Also contains: glyceryl stearate, caprylic/capric triglyceride, sucrose distearate, sucrose stearate, cetylstearyl alcohol, vegetable glycerine, trihydroxystearin, vegetable oil, hydrogenated vegetable oils, candelilla wax, xanthan gum, citric acid, tocopherols, sodium silicate, benzyl alcohol, potassium sorbate, sodium dehydroacetate, sunflower oil.*

*Organically Farmed Ingredient

WARNINGS: do not use in case of individual hypersensitivity or allergy to one or more components. If symptoms do not improve, seek medical advice. Store at room temperature, away from heat sources and light. Keep out of reach of children. The expiry date refers to the intact product, stored correctly.

EXPIRY DATE: see the expiry date stated on the package. Do not use after the expiry date indicated.

Upon opening, it is recommended to use the product within 4 months.

PACK FOR SALE

40 ml tube Contains endorectal applicator.

Last revision date: 05/2015

FOR TODAY AND TOMORROW'S HEALTH

Aboca is a Benefit Corporation and certified B Corp. We are committed to operating responsibly, sustainably and transparently towards the community and the environment. Impact report available at www.aboca.com

Aboca S.p.A. Società Agricola
Località Aboca, 20 - 52037 Sansepolcro (AR) - Italia - www.aboca.com
Company operating under certifications ISO 9001 and ISO 13485 for quality management system and ISO 14001 for environmental management system.

Certified
 Corporation

PACKAGING MATERIALS		
Carton	PAP 21	> Paperboard
Leaflet	PAP 22	> Paper
Tube	ALU 41	> Aluminium
Cap	PP 5	> Plastic
Cannula sachet	PP 4	> Plastic
Cannula cover	LDPE 4	> Plastic
Cannula	PP 5	> Plastic

For disposal, please check the regulations in your local area.

ALLEAR01Z-B