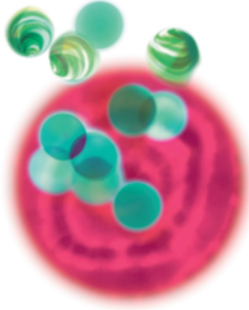
Oroben

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
Oroben

- mouth ulcers • gingivitis
- minor lesions
- daily oral hygiene

**Reduces pain, protects,
and promotes healing**

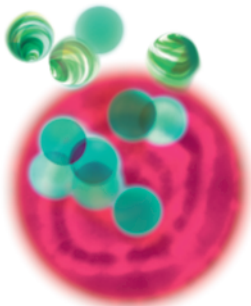with **RESINOX^{FP}**,
a natural molecular complex of
polysaccharides, resins and flavonoids

100%
NATURAL AND
BIODEGRADABLE FORMULA

gluten
free

WITH INGREDIENTS
FROM ORGANIC
FARMING

MEDICAL DEVICE Class IIa **CE** 0477

MOUTHWASH

EN **OroBen Mouthwash** is a medical device that reduces pain and promotes the healing of mouth ulcers, recurrent aphthous stomatitis, gum inflammation and minor oral lesions caused by (e.g.) dental treatment or the use of braces and dentures.

HOW IT WORKS: OroBen Mouthwash works thanks to **RESINOX FP**, a natural molecular complex developed by Aboca Research, composed of polysaccharides, resins and flavonoids which form an active barrier on the mucosa, with a dual action:

1. PROTECTIVE-HYDRATING

Forms a film that adheres to the lesion or inflamed area, limiting contact with external agents, pathogens or irritants that impede the natural healing and tissue regeneration process.

2. ANTIOXIDANT

Protects the mucosa from the irritation caused by free radicals.

Through these mechanisms, the product exerts an **indirect anti-inflammatory action** and promotes tissue repair, reducing healing time. In addition, the peppermint essential oil provides a refreshing effect, leaving a pleasant sensation of freshness in the mouth, which also helps to reduce halitosis that often accompanies gingivitis. With OroBen Mouthwash, it is possible to reach every area of the oral cavity by rinsing and gargling. It removes irritants, and thus helps to control the microenvironment, improve hygiene and protect the mouth.

INDICATIONS: specifically designed for the treatment of mouth ulcers, recurrent aphthous stomatitis and gum inflammation. It can also be used for minor oral lesions resulting from (e.g.) dental treatment or the use of braces and dentures. In cases of particular sensitivity, it can be used preventively to protect the oral mucosa, even for prolonged periods. Also suitable for children over 6 years of age. Non-hazardous if accidentally swallowed during use. The high concentration of functional substances may result in a slight deposit at the bottom of the bottle; a gentle shake is all that is needed to achieve a homogeneous dispersion.

INSTRUCTIONS FOR USE: shake the bottle before use. Measure approximately 10 ml of mouthwash using the measuring cup provided, and rinse and gargle for at least one minute, 2 to 3 times a day. The product can be used either pure or diluted in a little water. For use in children, it is recommended to dilute the product. Avoid food or drink for at least 15 minutes after use to allow the product to adhere to the mucosa. Close the bottle tightly after use and wash the measuring cup thoroughly.

COMPOSITION - Functional substances: **RESINOX FP**, a natural molecular complex of polysaccharides (from Aloe vera*), flavonoids (from Chamomile*) and resins (from Myrrh).

Also contains: Hydroalcoholic solution; Vegetable glycerine; Ethyl alcohol from organic wheat*, Peppermint essential oil; Natural flavourings; Licorice extract*; Xanthan gum.

*Ingredient from Organic Farming

Does not contain preservatives.

WARNINGS: do not use in case of individual hypersensitivity or allergy to one or more components. Store in a cool, dry place, away from heat sources and protected from light. Keep out of reach of children. The expiry date refers to the intact product, stored correctly. If symptoms persist, seek medical advice to rule out systemic or dental causes that require other types of treatment.

See the expiry date indicated on the pack. Do not use after the expiry date indicated. We recommend using the product within 6 months of opening.

PACKAGE: 150 ml glass bottle

Contains measuring cup

Hydrates 150 ml - Anhydrous ingredients 30 ml

Lic. UTF IT00PG - UTF A00035A - Alcohol 20% vol.

LAST REVISION DATE: 11/2018

 **Aboca S.p.A. Società Agricola**
Località Aboca, 20
52037 Sansepolcro (AR) - Italia
www.aboca.com

Certified

Corporation

 PACKAGING MATERIALS		
Carton box	PAP 21	> Paperboard
Leaflet	PAP 22	> Paper
Bottle	GL 72	> Glass
Cap	C/ALU 90	> Predominantly aluminium
Measuring cup	PP 5	> Plastic
For disposal, please check the regulations in your local area.		

ALLEA00001X-A